

MEDITELLE CLEANING INSTRUCTIONS:

Couches

Vinyl Upholstery

Vinyl is resistant to most mild acids, alkalis, drinks and household stains. Clean with a damp soapy cloth and rinse well with clean water. Do not use solvents, bleaches*, abrasives, synthetic detergents, wax polishes or aerosol sprays. Disinfectants may be used in dilution as specified by their manufacturers.

*Chlorine releasing agents – levels of 1 – 10000 parts per million should pose no problems as long as the material does not endure long soakings at the higher concentrations. Rinse well with clean water. These include Haz-Tabs, Chlor-Clean, Milton, Virusolve and Tristel and other disinfectants commonly used in hospital infection control programmes. Vinyl surfaces are Nadcc compatible.

Framework

Regular cleaning with warm soapy water and a soft cloth or anti-bacterial wipes. A mild disinfectant may be used in dilution. Do not use undiluted strong detergents, abrasives or solvent /chemical cleaners.

Chairs/ Stools

Vinyl Upholstery

Regular cleaning with warm soapy water followed by a clean water rinse. Heavy, ingrained soiling may be removed by scrubbing with a soft brush. Do not use strong detergents, solvent cleaners, bleaches, abrasives, polishes, aerosol sprays or chemical cleaners. Disinfectants may be used in dilution. Vinyl surfaces are Nadcc compatible.

Bases

Regular cleaning with warm soapy water and a soft cloth or anti-bacterial wipes. A mild disinfectant may be used in dilution. Do not use undiluted strong detergents, abrasives or solvent /chemical cleaners.

Trolleys

Regular cleaning with warm soapy water and a soft cloth or anti-bacterial wipes. A mild disinfectant may be used in dilution. Do not use undiluted strong detergents, abrasives or solvent /chemical cleaners.



Meditelle Group, Manufacturing Centre, Moor Lane, Witton. Birmingham. B6 7HH.

Tel: 0121 332 1850 Web: www.meditelle.co.uk | www.meditelle-dental.co.uk | www.meditelle-first-aid.co.uk