

These instructions can be applied to any of our operator stool or chair models.

To Disassemble

• Step 1 - Remove Pad from the gas column:

Turn the seat upside down. Holding the seat by the gas column gently tap the metal plate that surrounds the gas column on the underside of the seat pad using a mallet / hammer. Pic 1.

Continue to tap until the gas column works loose and the seat pad becomes disconnected. Pic 2.

• Step 2 - Remove Base from the gas column:

You will now be left with the base connected to the gas column.

Turn the base upside down to locate the gas column protruding through the underside centre of the base. Pic 3.

Using a mallet / hammer gently tap around the edge of the protruding gas column (avoiding the central pin) until the column works free of the base. Pic 4.

Please contact us for further technical assistance if required.

