

General Cleaning Instructions

Couches & Chairs

- Framework

Regular cleaning with warm soapy water and a soft cloth or anti-bacterial wipes. A mild disinfectant may be used in dilution. Do not use undiluted strong detergents, abrasives or solvent /chemical cleaners.

We also recommend to periodically oil the axles and moving parts with a liquid grease spray e.g. WD40

- Aluminium

Aluminium should be cleaned using a soft damp non-abrasive cloth and polished completely dry. Exposure to wet or humid conditions could result in discolouration or corrosion. Keep all surfaces dry at all times. Bleach, synthetic detergents, solvents, wax polishes or aerosol sprays should not be used.

- Electrical components

Keep the unit free from any dust and grease by light dusting with a clean dry cloth. No electrical components should be exposed to any form of liquid or humid conditions. Plastic casings of actuators or handsets may be cleaned using a soft damp non-abrasive cloth and polished dry. Bleach, synthetic detergents, solvents, wax polishes or aerosol sprays should not be used.

Note: For safety reasons any electrical equipment must be disconnected from the mains before cleaning.

Operator Chairs & Stools

- Bases

Regular cleaning with warm soapy water and a soft cloth or anti-bacterial wipes. A mild disinfectant may be used in dilution. Do not use undiluted strong detergents, abrasives or solvent /chemical cleaners.

Trolleys

- Steel Coated

Regular cleaning with warm soapy water and a soft cloth or anti-bacterial wipes. A mild disinfectant may be used in dilution. Do not use undiluted strong detergents, abrasives or solvent /chemical cleaners.

- Stainless Steel

Stainless steel should be cleaned using a soft damp cloth and polished completely dry. Do not use abrasive cloth. Constant use of certain cleaning products such as Iodine may result in permanent damage to the metal surface and should not be used. Exposure to wet or humid conditions could result in discolouration or rust marks even on 'stainless' steel. Keep all surfaces dry at all times. Bleach, synthetic detergents, solvents, wax polishes or aerosol sprays should not be used.

Vinyl - To Clean & Disinfect

A mild detergent and water are the best day-to-day cleaning solutions for your vinyl fabric. With a clean sponge or a soft cloth and soapy luke-warm water, wipe the vinyl to remove dirt or any surface stains. Do this on a consistent basis so that dirt and grime don't accumulate and damage the fabric. Clean the area in circular motions. If disinfection is necessary, clean with a solution of disinfectant and a damp cloth. Never use harsh cleaning solvents or cleansers on your vinyl fabric, particularly cleaners that are abrasive. Check the cleaning products instructions for proper use of the product and if using for the first time test your cleaner on a sample patch before applying to a more visible part of the fabric.

Our fabrics can be used with all NHS approved cleaning solutions such as Haz-Tabs, Chlor-Clean, Milton, Virusolve, Tristel and Vernacare Tuffie 5 Sanitising Wipes. If using bleach a solution of 5% bleach to 95% water is acceptable.

Always follow any wet cleaning or removal of any stains with a clean cold water rinse and dry with a soft cloth or towel.

T: 0121 332 1850

E: enquiry@meditelle.co.uk

W: meditelle-dental.co.uk

