

## PAGET / WINDSOR MODELS

Your portable couch will be supplied fully assembled. Before using your couch please familiarise yourself with its workings:

### To Open

- Stand the folded couch upright on the protective rubber pads.
- Release fastener catch at the end of the couch.
- Open the couch top out until fully extended. The central legs will automatically unfold to position.
- Extend both outer sets of legs fully.
- Straighten the four leg stays, to lock each leg in place.

### To Stand Couch Upright

- Stand facing the upholstery side of couch, lift the couch upwards and away from you onto its legs.

### To Close

- Turn the couch over onto its side to rest on the protective rubber pads.
- Release the stays by folding inwards.
- Fold end legs into the frame.
- Bring together each half of the couch and secure the fastener catch.

### Backrest Adjustment

**Pole Mechanism** (certain models only) - Position the backrest support pole through the hole in the swing plate attached to the frame. To adjust backrest, lift swing plate to release the pole and move back section to desired position, release plate to secure in place.

*Note: Before closing couch ensure backrest pole is fully taken out of the swing plate.*

**Ratchet Mechanism** (certain models only) - Backrest can be raised / lowered. The multi-positioning ratchet mechanism will locate in the next position as the back section is moved up. To set a lower position the back section must be raised completely to vertical before moving down through the ratchet series.

### Upholstery Care

Clean with warm, soapy water and a clean soft cloth. Rinse well with clean water and dry with a soft cloth. Disinfectants may be used in dilution as specified by the manufacturers.

Our fabrics can be used with NHS approved cleaning solutions such as Haz-Tabs, Chlor-Clean, Milton, Virusolve, Tristel and Vernacare Tuffie 5 Sanitising Wipes. If using Bleach a 5% to 95% water solution is acceptable. DO NOT use solvents, abrasives, synthetic detergents, wax polishes or aerosol sprays. Never use boiling water or steam cleaners.

### Safety Information

Meditelle couches meet relevant British safety regulations within British Safety Standards BS3379/BS5852 and BS EN 60601-1:1190 and BS EN60601-2-38 and The Furniture & Fire Safety Regulations S11324:1989.

For full technical details see website.



Safety Note: Do NOT sit on the extremities of the couch. Distribute weight evenly.

