

## DUNBAR DUN5100-DUN5195 / BLENHEIM BSC316

### You should have the following component parts:

- Upholstered Top Section complete with frame (back ratchets already attached if applicable)
- 4 x Leg Pieces

### To Assemble

- Step 1 - Carefully place the couch face down on the floor, so the under frame is facing upwards.
- Step 2 - Identify the threaded bar at the top of each leg. In turn, insert the threaded bar end of each leg into the round holes set in metal blocks positioned at the four corners of the frame. Turn legs clockwise to secure into place. **Take care not to overtighten.**
- Step 3 - Carefully lift the couch onto its side, then onto its legs.

### Towel Roll Holder (if applicable)

This will be pre-fitted onto your couch at time of manufacture and does not require assembly.

### Back Section

The ratchet mechanism will locate in the next position as the backrest is moved up. To set to a lower position the backrest must be raised completely before being moved down through the ratchet series.

## BLENHEIM BSC300 / MALORY MAL5400

### You should have the following component parts:

- Upholstered Top Section complete with frame (backrest pole already attached)
- 2 x Leg Bracing Stays
- *If ordered with step - 1x Step with x2 bolts inserted / 1 x Connecting Bar with x2 bolts inserted*

### To Assemble

- Step 1 - Carefully place the couch face down on the floor, so the under frame is facing upwards.
- Step 2 - The legs will be folded down inside the bed frame. Lift both legs outwards from the centre and raise up to 90°.
- Step 3 - Unscrew bolt from each leg and bolt from centre of frame. Align each bracing stay with the corresponding bolt holes in the legs and centre frame and insert bolts into the x3 holes leaving a loose fitting at this stage.
- Step 4 - When both legs are fitted in place tighten bolts up to secure. **Take care not to overtighten.**
- Step 5 - Carefully lift the couch onto its side, then onto its legs.
- Step 6 - When upright, lift backrest and insert the attached pole through hole in the swinger tab plate located in centre of frame under the back section.

### To Fit Step (on MAL5400 only)

- Step 1 - When couch is upright, decide which side you want to locate the step. Take long metal connecting bar to chosen side of couch and secure bar to the legs at each end using x2 bolts supplied (bolts will be attached to frame).
- Step 2 - Attach step by aligning the steps hinge holes with the holes in the bar and secure using x2 bolts supplied (bolts will be attached to frame).

### Towel Roll Holder (if applicable)

This will be pre-fitted onto your couch at time of manufacture and does not require assembly.

### Back Section

The multi-positioning backrest is adjustable by releasing the backrest swinger tab and lifting the backrest section of couch. When the back is in the required position, release tab and the backrest pole will automatically hold in desired position.

### Upholstery Care

Clean with warm, soapy water and a clean soft cloth. Rinse well with clean water and dry with a soft cloth. Disinfectants may be used in dilution as specified by the manufacturers. Our fabrics can be used with NHS approved cleaning solutions such as Haz-Tabs, Chlor-Clean, Milton, Virusolve, Tristel and Vernacare Tuffie 5 Sanitising Wipes. If using Bleach a 5% to 95% water solution is acceptable. **DO NOT** use solvents, abrasives, synthetic detergents, wax polishes or aerosol sprays. Never use boiling water or steam cleaners.

### Safety Information

Meditelle couches meet relevant British safety regulations within British Safety Standards BS3379/BS5852 and BS EN 60601-1:1190 and BS EN60601-2-38 and The Furniture & Fire Safety Regulations S11324:1989.

For full technical details see website.



**Safety Note: Do NOT sit on the extremities of the couch. Distribute weight evenly.**





## BLENHEIM BSC330

### You should have the following component parts:

- Upholstered Top Section complete with pair side rails (with x4 bolts inserted)
- 1 x Backrest Pole (backrest adjusting bar and pole bracket already attached to couch)
- 2 x Leg Pieces
- 1 x Stability Bracing Rail

### To Assemble

- Step 1 - Carefully place the couch face down on the floor, so the under frame is facing upwards. Take the x4 bolts out of the side rails.
- Step 2 - Place the two leg pieces across the side rails, ensuring that the guide tags on the legs are facing inwards. Align the bolt holes in the legs and side rails and insert the bolts into the x4 holes leaving a loose fit at this stage, to allow the middle stability rail to be inserted. Pic 1.
- Step 3 - Take the bolts out of the ends of the stability rail. Align the stability bracing rail with the guide tags. Secure in place by inserting the bolts through from the outside of the leg into the rail. Return to the x4 bolts along the side rails and tighten to secure the legs in place. Pic 2.
- **Take care not to overtighten.**
- Step 4 - To fit the backrest pole first ensure that the tab on the adjusting bar is facing inwards. Take the bolt out of the backrest pole and place the pole through hole in the bar tab and into the pole bracket. When in place secure by fitting the bolt through the bracket. Pic 1.
- Step 5 - Carefully lift the couch onto its side, then onto its legs.
- Step 6 - Use the adjusting feet on each corner to level the couch on an uneven floor.

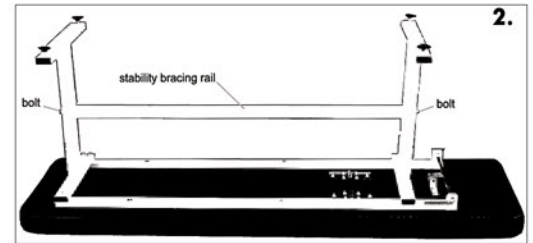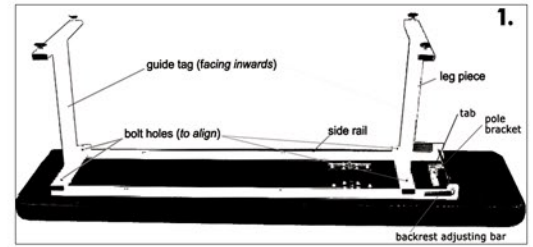

### Towel Roll Holder (if applicable)

This will be pre-fitted onto your couch at time of manufacture and does not require assembly.

### Back Section

The multi-positioning backrest is adjustable by releasing the backrest adjusting bar and lifting the top head section of couch. When the back is in the required position, release the bar and the backrest pole will automatically be held in position by the tab plate.



## MARSDEN MAR5300 / BLENHEIM BSC350

### You should have the following component parts:

- Upholstered Top Section complete with frame (with x4 bolts inserted)
- 1 x Backrest Pole (backrest adjusting bar and pole bracket already attached to couch)
- 2 x Leg Pieces
- 2 x Stability Bracing Rails

### To Assemble

- Step 1 - Carefully place the couch face down on the floor, so the under frame is facing upwards. Take the x4 bolts out of the framework on each corner.
- Step 2 - Place the two leg pieces across the frame at each end, ensuring that the guide tags on the legs are facing inwards. Align the bolt holes in the legs and frame and insert the bolts into the x4 holes leaving a loose fit at this stage, to allow the middle stability rail to be inserted. Pic 1.
- Step 3 - Take the bolts out of the ends of the double stability rails. Align the stability bracing rails with the guide tags. Secure in place by inserting the bolts through from the outside of the leg into the rails. Tighten bolts to secure in place. Pic 2. **Take care not to overtighten.**
- Step 4 - To fit the backrest pole first ensure that the tab on the adjusting bar is facing inwards. Take the bolt out of the backrest pole and place the pole through the hole in the bar tab and into the pole bracket. When in place secure by fitting the bolt through the bracket. Pic 1.
- Step 5 - Carefully lift the couch onto its side, then onto its legs.
- Step 6 - Use the adjusting feet on each corner to level the couch on an uneven floor.

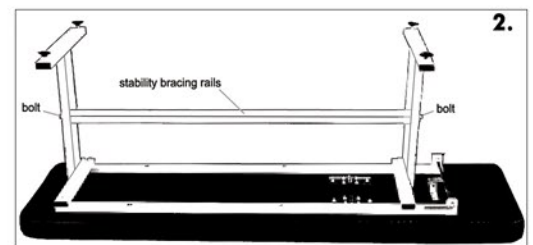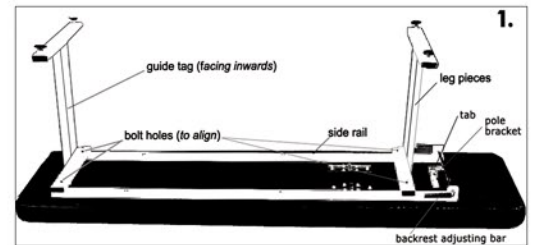

### Towel Roll Holder (if applicable)

This will be pre-fitted onto your couch at time of manufacture and does not require assembly.

### Back Section

The multi-positioning backrest is adjustable by releasing the backrest adjusting bar and lifting the top head section of couch. When the back is in the required position, release the bar and the backrest pole will automatically be held in position by the tab plate.

**For Upholstery Care & Safety Information please see overleaf.**